

THIS PACKAGE INSERT CONTAINS IMPORTANT PRODUCT USE AND SAFETY INFORMATION. PLEASE READ CAREFULLY AND RETAIN FOR FUTURE REFERENCE.

# SOLO CARE AQUA®

**INDICATIONS / ACTIONS** | The all-in-one contact lens care solution for cleaning, rinsing, disinfecting, storing and lubricating soft (hydrophilic) contact lenses, including silicone hydrogel lenses, as recommended by your eye care professional. SOLOCARE AQUA® All-In-One Solution is also indicated for daily protein removal when lenses are soaked for at least four (4) hours. No separate cleaner is required when used daily.

The SOLOCARE AQUA® All-In-One Solution MicroBlock® Lens Case is indicated for storage of soft (hydrophilic) contact lenses, including silicone hydrogel lenses. The antibacterial properties of the case are intended to reduce the growth of micro-organisms on the lens case.

**INGREDIENTS** | SOLOCARE AQUA® All-In-One Solution is a sterile isotonic aqueous solution containing polyhexanide 0.0001% (w/v).

One (1) SOLOCARE AQUA® All-In-One Solution MicroBlock® Lens Case is included.

**DIRECTIONS FOR USE** | To ensure proper disinfection of your lenses you must follow the instructions completely. Do not skip any steps.

- (1) Wash and rinse your hands and dry with a clean, lint-free towel before handling your lenses.
- (2) Apply at least 3 drops of SOLOCARE AQUA® All-In-One Solution on each side of the lens and gently rub the lens for at least 10 seconds.
- (3) Rinse both sides of the lens with SOLOCARE AQUA® All-In-One Solution for at least 10 seconds.
- (4) Place the lens in the empty MicroBlock® lens case and fill to the line with fresh SOLOCARE AQUA® All-In-One Solution making sure to cover the lens. Repeat the above procedure with the second lens. Replace caps and screw closed.
- (5) Allow your lenses to soak for at least 5 minutes to overnight.  
After soaking, lenses are ready to use. If needed, fresh SOLOCARE AQUA® All-In-One Solution may be used to rinse the lens prior to insertion.
- (6) Always empty your MicroBlock® lens case and discard used solution after each disinfection cycle. Then rinse your MicroBlock® lens case with fresh SOLOCARE AQUA® All-In-One Solution and recap.

**STORAGE OF LENSES, SOLUTION AND LENS CASE** | Lenses may be stored in SOLOCARE AQUA® All-In-One Solution in the unopened lens case for up to 30 days after disinfection (soaking). Every 30 days, discard the old solution from the lens case, use fresh solution and repeat the rub, rinse and soak procedures described above, before wearing your lenses again.

Store both the SOLOCARE AQUA® All In One Solution and the MicroBlock® lens case at room temperature (15°C to 30°C).

**CONTRAINDICATIONS** (Reasons not to use) |

If you are allergic to any ingredient in this solution or to silver or other metals, do not use.

**WARNINGS**

- Always use fresh SOLOCARE AQUA® All-In-One Solution each time you disinfect. Never reuse the solution or add fresh solution to used solution.
- Only use SOLOCARE AQUA® All-In-One Solution with your lens and lens case. Do not use tap water, bottled water or saliva.
- Always use SOLOCARE AQUA® All In One Solution with the MicroBlock® lens case. Do not substitute with components from any other lens care system.
- If irritation develops with the use of this product, discontinue use and consult your eye care professional.
- Do not touch tip of the bottle to any surface since this may contaminate the solution. Always keep bottle tightly closed.

Failure to follow your eye care professional's instructions and all product labeling directions may result in serious eye infections or loss of vision. Prompt diagnosis and treatment are essential; therefore if you experience any unusual symptoms or adverse reactions, listed below, immediately remove your lenses and contact your eye care professional.

**PRECAUTIONS**

- Replace your MicroBlock® lens case with every purchase of SOLOCARE AQUA® All-In-One Solution or as recommended by your eye care professional
- Use SOLOCARE AQUA® All-In-One Solution before expiration date marked on the bottle.
- Keep out of reach of children.
- Data indicate that lens cases are a significant source of microbial contamination. Therefore, proper lens case cleaning is essential for minimizing the risk of contact lens related eye infections. Refer to the Directions for Use for proper lens case care.
- The MicroBlock® lens case may slightly discolour over time; this discolouration does not affect the antibacterial properties of the lens case.
- Never use SOLOCARE AQUA® All-In-One Solution for heat disinfection.

**ADVERSE REACTIONS** (Possible problems and what to do) |

The following problems may occur: eyes sting, burn or itch (irritation), comfort is less than when lens was first placed on eye, feeling of something in the eye (foreign body, scratched area), excessive watering (tearing) of the eye, unusual eye secretions, redness of the eyes, reduced sharpness of vision (poor visual acuity), blurred vision, rainbows or halos around objects, sensitivity to light (photophobia) or dry eyes.

**If you notice any of the above: IMMEDIATELY REMOVE YOUR LENSES** |

- If the discomfort or problem stops, then look closely at the lens
- If the lens is in any way damaged, DO NOT put the lens back on your eye. Place the lens in the storage case and contact your eye care professional
- If the lens has dirt, an eyelash, or other foreign body on it, or the problem stops and the lens appears undamaged, thoroughly clean, rinse and disinfect the lens; then reinsert it.
- If the problem continues, immediately remove the lens and promptly consult your eye care professional.

**Professionals** | All adverse reactions observed using SOLOCARE AQUA® All-I

Menicon Co., Ltd

3-21-19, Aoi, Naka-ku, Nagoya, 460-0006 Japan

**HOW SUPPLIED** | SOLOCARE AQUA® All-In-One Solution is a sterile solution supplied in 90ml and 360ml recloseable plastic containers. Bottles and cartons are marked with a lot number and expiration date.

One (1) SOLOCARE AQUA® All-In-One Solution MicroBlock® Lens Case is provided with each bottle of SOLOCARE AQUA® All In One Solution.

SOLOCARE AQUA, MicroBlock are registered trademarks of Novartis AG

6301/05-1