This "Instructions for Use" contains important information. Please read carefully and retain for future reference.

PROGENT

Menicon Progent Protein Remover for Rigid Gas Permeable Contact Lenses("PROGENT").

Manufacturer
Menicon Nect Co., Ltd
390, Ichibagi-cho, Nishi-ku, Nagoya
452-0805, Japan


INDICATIONS

Protein remover for all rigid gas permeable lenses.

DIRECTIONS

The PROGENT treatment is recommended every two weeks.

- ① Place the lenses into PROGENT Vial caps in its appropriate right or left holder: R = right, L = left, for intensive cleaning.
- ② Open the doses carefully without touching the opening and avoid contact with clothing.
- ③ Fill the PROGENT Vial with A solution then B solution.
- ④ Close the PROGENT Vial cap securely. Gently shake it a few times.
- ⑤ Soak the lenses in PROGENT for 30 minutes maximum.
- ⑥ Remove the PROGENT Vial cap, empty its content. Rinse the lenses in the holder thoroughly with a rinsing solution.
- ⑦ Then clean the lenses thoroughly with a regular solution and rinse them. The lenses may then be worn or stored in the storage case.
- ⑧ Rinse the PROGENT Vial and its cap thoroughly with the rinsing and regular solution and let them air dry.

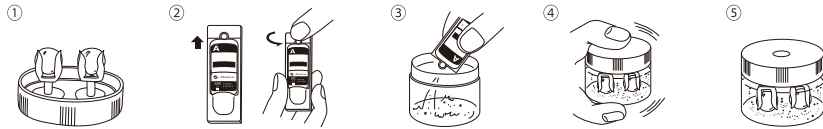WARNINGS

Do not apply PROGENT to the eyes, the nose and the mouth. In case of misuse, rinse your eyes immediately and thoroughly with water and consult your eye care practitioner.

- It is essential to rinse the lenses abundantly with a rinsing and regular solution after soaking them in PROGENT solution.
- To help prevent infections, do not rinse with tap water.
- Keep out of reach of children.

PRECAUTIONS

- Always wash and rinse your hands before handling contact lenses.
- Use only freshly mixed PROGENT solution.
- Do not use PROGENT A and B solutions separately.
- Single use doses. Discard the solution and the doses after each use. Never reuse the solution.
- Do not use PROGENT A and B after the expiry date printed on the single dose and outer carton.
- Do not soak lenses in PROGENT for over 30 minutes. Lenses might discolor. However, this will not affect their quality.
- Use only PROGENT Vial. Do not use any other container for solution mixing and lens cleaning with PROGENT.
- Important: do not swallow.
- Store at room temperature.
- Store out of direct sunlight.
- Avoid contact with the skin and do not spill PROGENT on your clothes. It may bleach them. Wash clothes immediately.

CONTRAINDICATIONS

- Do not use for soft lenses.
- Do not use PROGENT if you are allergic to any ingredient in the product.

ADVERSE REACTIONS (PROBLEM AND WHAT TO DO)

The following problems may occur:

eyes sting, burn or itch (irritation), comfort is less than when lens was first placed on the eye, feeling of something in the eye (foreign body, scratched area), excessive watering (tearing) of the eye, unusual eye secretions, redness of the eye, reduced sharpness of vision (poor visual acuity), blurred vision, rainbows or halos around objects, sensitivity to light (photophobia), or dry eyes.

If you notice any of the above, immediately remove your lenses.

- If the lens is in any way damaged do not reinsert.
- If the lens does not appear to be damaged and there is not discomfort or a problem, thoroughly clean, rinse and disinfect the lens, then reinsert it. If any discomfort or problem occurs, immediately remove the lens and consult your eye care professional for identification of the problem and, if necessary, obtain treatment to avoid serious eye damage.
- If PROGENT solution is accidentally put into the eye or nose, wash thoroughly for about 1 minute with sterile saline solution or sterile purified water or another appropriate rinsing solution. Consult your eye care professional or other healthcare practitioner and, if necessary, obtain treatment to avoid serious damage.
- If PROGENT solution is accidentally swallowed, drink a large quantity of water. Seek immediate professional medical assistance or contact a poison control center.
- If PROGENT solution is accidentally spilled on the skin, wash thoroughly with running tap water for about 1 minute.

PACKAGING

PROGENT is supplied in 7-treatment or 1-treatment packages.

The 7-treatment package consists of seven 5 mL containers of PROGENT A solution, seven 5mL containers of PROGENT B solution, and one PROGENT Vial.

The 1-treatment package consists of one 5 mL container of PROGENT A solution, one 5 mL container of PROGENT B solution, and one PROGENT Vial.

PROGENT A and PROGENT B containers are for single use only. The outer carton and each solution container are printed with a lot number and expiration date.

COMPOSITION

Solution A: Sodium hypochlorite (NaClO), sodium carbonate, sodium hydroxide, purified water; 5 mL

Solution B: Potassium bromide (KBr), sodium carbonate, purified water; 5mL

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