**MENISOFT Patient Instructions. Monthly replacement soft hydrophilic contact lenses**

Read this notice carefully and keep this information for future use.

**INTRODUCTION**

This material is a non ionic copolymer of N,N-dimethyl acrylamide and N-vinyl-2-pyrrolidone with 72 % water (mipacron A), ending in tri-aqua-green.

ach stete MENISOFT contact lens is immersed with buffered saline solution in individual foil sealed tamper-evident plastic container. 3 contact lenses containers are packed in a multi pack box.

**NOTIFICATIONS AND WEARING RESTRICTIONS**

MENISOFT contact lenses are intended for daily wear and use by the person who has no astigmatism, no near-sightedness or farsightedness and is instructed by your eye care professional.

**Do not wear your MENISOFT daily wear contact lenses while sleeping.**

**CONTRAINDICATIONS**

You should not use MENISOFT contact lenses when any of the following conditions exist:

- Acute or subacute inflammation or infection of the anterior chamber of the eye  
Any eye condition in which there is any threat to the cornea, the conjunctiva or eyelids

Suspected insensitivity of bacterial secretion (dry eyes).

Conical hypoplasia (induced corneal sensitivity), if not abrogated.

Any systemic disease that may affect the eye or be aggravated by wearing contact lenses.

Allergic reactions of ocular surfaces or adhesia that may be induced or exaggerated by wearing contact lenses or use of care solutions.

Any dry eye syndrome.

If you are pregnant or lactating.

Incomplete healing following eye surgery.

**WARNINGS**

You should be advised of the following warnings pertaining to contact lens wear:

Problems with contact lenses and lens care products could result in **serious injury** to the eye. It is essential that you follow your eye care professionals directions and instructions for the safe use of lenses and care products, including the care case. Eye problems, including corneal ulcers, can occur rapidly and lead to **loss of vision**.

MENISOFT daily wear lenses are not intended for overnight wear, and you should not wear lenses while sleeping. Clinical studies have shown that the risk of some adverse reactions is increased when these lenses are worn overnight.

If you experience **eye discomfort**, excessive tearing, vision changes, or redness of the eye, you should **immediately remove your lenses** and promptly contact your eye care professional.

**REACTIONS ADVERSE**

Any of the following symptoms may occur: • eyes burning, itching or other eye pain; • comfort is less than when lens was first placed on eye; feeling of dryness in your eye such as foreign body or scratch感; • excessive watering (tearing) of your eyes; • eyes feel sore; • eyes feel redness, reduced tear production (poor visual acuity); blurred vision, halos, photophobia; • dry eyes ... **you should immediately remove your lenses.**

The problem stops, look closely at the lens. If the lens is in any way damaged, do not put it back on your eye. Place the lens in the storage and contact lens case, rinse the lens with water and then place the contact lens case on your head or in your pocket. If the problem continues, **you should immediately remove the lenses and consult your eye care professional.** When any of the above problems occur, a serious condition such as infection, corneal ulcer, neovascularization or iritis may be present. You should **keep lens off your eye and seek immediate professional identification of the problem and prompt treatment to avoid serious eye damage.**

**PERSONAL CLEANLINESS FOR LENS HANDLING**

Wash, rinse and dry your hands before handling your lenses.

**Opening the multi-pack carton and lens containers:**

A small opening may be on the same side for both lenses, check the mark box indicating which lens is for your left and right eye. Be sure to open and use the correct lens for each eye.

Open the opening and pull up to break the seal (fig. 1).

Once you will find 3 lenses in 3 separate containers designed to maintain sterility.

remove the lenses from the multi pack box.

Do not use the lens if the sterile package has been opened or if the tamper-evident seal is damaged

o open a new lens container

Peel back lid to reveal the lens (fig. 3). By stabilizing the lens container on the tabletop, you will minimize the possibility of a sudden splash.

Carefully remove the lens from the container (fig. 4) and inspect the lens following the handling instructions.

o close the multi-pack for storage, just tuck in the flap.

adding the lenses

develop the habit of always working with the same lens first to avoid mix-ups.

remove the lens from its storage case and examine to be sure that it is moist, clean and free of any nick or tears.

using the lens on the eye

our eye care professional should provide you with a detailed method but here is a standard procedure to refer:

acc the lens on the tip of the forefinger with your dominant hand, concave side up

Make sure the lens is not turned inside out by placing it on your index finger and checking its profile.

old your eyelids wide apart with the thumb and forefinger of the other hand.

keep your eye wide open and slowly bring your lens up to the eye looking straight ahead. Gently place the lens on the cornea. Do not press the lens against your eye.

developing a habit and being gentle

repeat this procedure for the other lens

o if after placement of the lens, your vision is blurred, check for the following :

old a mirror above the eye, keep the head straight, look upwards to see the eye in the mirror;

old down the lower lid with the middle finger of your removal hand and place your index finger on the lower edge of the lens.

old the lens down to the lower white part of the eye.

Diligently, place the lens onto the thumb and forefinger.

remove the lens

repeat the same procedure for the other lens

**ARE FOR YOUR LENSES**

is important to follow the lens care procedure recommended by your eye care professional.

When you first receive your lenses, you should be able to put and remove lenses or should have someone else who can place and remove the lenses for you. You will be provided with recommended cleaning and disinfecting regimen. Heat disinfection is not recommended.

your eye care professional should carefully instruct you about appropriate and adequate procedures and products for your use. Notice inside the box are also when you a lens for the first time.

The lens is a contact lens.

In case of cosmetics or oils the lens clean, rinse and place on the eye again.

The lens is the wrong eye.

The lens is inside.

your vision is blurry and after checking the above possibilities, remove both lenses and consult your eye care professional.

**Remove the lens:**

Wash, rinse and dry your hands.

old a mirror above the eye, keep the head straight, look upwards to see the eye in the mirror;

old down the lower lid with the middle finger of your removal hand and place your index finger on the lower edge of the lens.

old the lens down to the lower white part of the eye.

Diligently, place the lens onto the thumb and forefinger.

remove the lens

repeat the same procedure for the other lens

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It is recommended that you wear your lenses completely immersed in the recommended storage solution when the lenses are not being worn.

Always keep the lenses completely immersed in the recommended storage solution when the lenses are not being worn.

Lens cases must be cleaned, rinsed with solutions recommended by your eye care professional and placed at regular intervals.

**recautions :**

Do not get cosmetics, lotions, creams... or gels in your eyes or on your lenses.

It is best to remove the lenses before make-up. Do not touch the lens with fingernails.

The lens sticks on your eye, you should apply a few drops of the recommended lubricant or rewetting solution directly to the eye and wait until it turns to soft state. After that, clear lens and dislodge the lens.

Avoid all contact with irritant vapors and fumes while wearing lenses.

If you remove the lenses completely immersed in the recommended storage solution when the lenses are not being worn.

Always keep the lenses completely immersed in the recommended storage solution when the lenses are not being worn.

Lens cases must be cleaned, rinsed with solutions recommended by your eye care professional and placed at regular intervals.

**Lenses prescribed in a planned replacement program should be thrown away after the recommended wearing period.**

**do never wear lenses beyond the period recommended by your eye care professional. Always discard lenses after their use with any contact lens, follow-up visits are necessary to assure the continuing health of your eyes.**

Carefully follow all the recommendations in this MENISOFT patient instructions and those prescribed by your eye care professional.

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**do not wear your MENISOFT daily wear and monthly replacement.** The lenses may be removed for routine cleaning and sanitization as instructed by your eye care professional.

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