

Fitting rules

	$\emptyset_T$	$K-K' < 0.20 \text{ mm}$	$0.20 \text{ mm} \leq K-K' \leq 0.40 \text{ mm}$	$0.40 \text{ mm} < K-K' \leq 0.50 \text{ mm}$
TORIC	Menicon Z BTC 9.10 9.60 10.10			Flattest BC = K Steepest BC = $K'+0.15$
	Menicon Z BT 9.10 9.60 10.10			Flattest BC = K Steepest BC = $K'+0.15$
	Menicon Z FT 9.10 9.60 10.10	BC = $K + 0.05$	BC = K	
MULTIFOCAL	Menifocal Z 9.60 9.80	BC = $K + 0.05$	BC = K	
	Menicon Z Executive 9.10 9.60 10.10	BC = $K + 0.05$	BC = K	
	Menicon Z Progressive 9.10 9.60 10.10	BC = $K + 0.05$	BC = K	
	Menicon Z Progressive BTC 9.10 9.60 10.10			Flattest BC = K Steepest BC = $K'+0.15$

K = flattest corneal radius K' = steepest corneal radius

MeniCare Plus (daily care) associated with Progent (weekly)

ORDERS AND ENQUIRIES

Tél. : 08003161507 - Fax : 08003161506

Menicon Holdings Europe - 104 rue Martre - 92583 Clichy Cedex - www.menicon.com